SpeediCath® Standard

Ready to use coated catheter.

Female/ Male

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Mujer/Hombre

How to use SpeediCath Standard

Begin by washing your hands. It is important to avoid transferring bacteria from your hands to the catheter.

The packaging contains the coated catheter in a saline solution. Before you attempt to open the catheter ensure you are completely ready to proceed with the catheterisation.

If the catheter has been stored vertically, turn it to a horizontal position to make sure that the saline solution is distributed evenly, ensuring that the coating is fully saturated. A full saturation ensures optimum lubrication.

1

Inspect the catheter before use. Do not use the product if the device or packaging is damaged.

Use the sticker on the back to attach the packaging to a smooth, dry horizontal surface. Open the packaging by pulling the ring towards you and then down. Wash your hands and the area around the urethral orifice. Take out the SpeediCath Standard catheter, which is now ready to use. Inspect the catheter before use. Do not use the product if the device or packaging is damaged.

Men

2a

Lift your penis with one hand to straighten your urethra. Use your other hand to insert the catheter into the urethra approx. 2 cm at a time. Just before the catheter reaches the bladder, you may feel a slight resistance from the sphincter. Do not squeeze the penis, as this may make inserting the catheter more difficult.

Tiemann/Coude catheter (curved tip) only:

Before you begin insertion, make sure the curved tip of the catheter is pointing upwards. The Tiemann indicator (raised ridge) on the connector is designed to help you orient the catheter correctly. When holding the catheter horizontally with the indicator facing upwards, the tip of the catheter will also be pointing upwards.

Women

2b

Part your labia with one hand to expose the urethral orifice. Use your other hand to insert the catheter gently into the urethra, approx. 2 cm at a time. Just before the catheter reaches the bladder, you may feel a slight resistance from the sphincter.

Continue to insert the catheter into the urethra until urine starts to flow. If there is no toilet nearby, you can connect the catheter to a urine bag or other collecting device.

3

When the urine stops flowing, ease out the catheter 2 to 3 cm. If the flow of urine restarts, wait a few seconds before easing out the catheter another 1 to 2 cm. When the bladder is completely empty, slowly remove the catheter. The product is intended for single use only and should be disposed of with normal household waste. Do not flush the product down the toilet.

Instructions for use Instrucciones de uso

23315405 Version 4

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved.© [2014-02-17]

Manufacturer: Coloplast A/S Høtveddam 1, 3050 Humlebaek, Denmark
Distributed by: Coloplast Corp., Minneapolis, MN, 55411, USA

Product Evaluation

For customer service or to return product, please call: 1-800-533-0464 in USA.

Prescription device

Caution: US Federal law restricts this device to sale by or on the order of a physician. Rx

Only

Indication

Urinary catheter for intermittent use.

The catheter is indicated for use by patients with chronic urine retention and patients with a post void residual volume (PVR) due to neurogenic and non-neurogenic voiding dysfunction. The catheter is inserted into the urethra to reach the bladder allowing the urine to drain.

Information

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

The catheter can be exposed to extreme temperatures (below 0°C and up to 60°C) for up to 24 hours without the catheter being damaged.

The catheter may be kept rolled up for up to 3 hours without being damaged.

The solution in which the catheter is stored is harmless. However, the solution may stain. Catheterization should take place as quickly as possible after opening the package. The catheter should always be removed when the bladder has been emptied.

Caution

Do not use if package is damaged

Contact your health care professional before you perform self-catheterization for the first time.

Contact your health care professional if you:

- experience symptoms such as fever, discomfort when emptying the bladder, a frequent urge to urinate or blood in the urine.

Federal (USA) law restricts this device to sale by or on the order of a physician.

Warning

Reusing this single-use product may create a potential harm to the user. Reprocessing, washing, disinfection, and sterilization may compromise product characteristics, causing additional risk of physical harm to or infection of the user.

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Fabricante: Coloplast A/S Holteedam 1 3050 Humlebaek Denmark