



INVISICLOSE™

HOW TO USE THE INVISICLOSE™ CLIPLESS TAIL CLOSURE



Lock-it-Pocket™




Two outlet
end strips




Two interlocking
closures


1 CLOSING THE POUCH

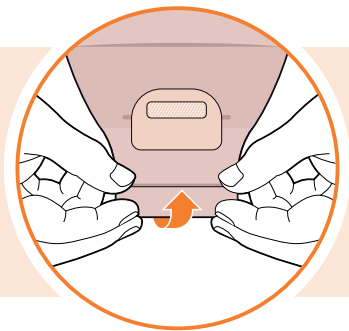

Step 1: Holding the outlet end strips, fold the end up towards you until the interlocking closures line up.

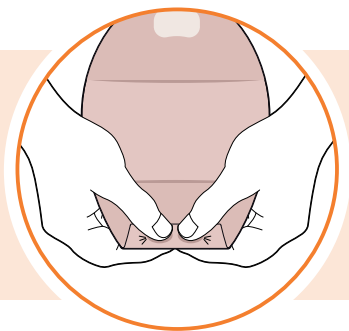

Step 2: Press the interlocking closures together across the width of the tail. Press and pinch across the closure until you feel them close.

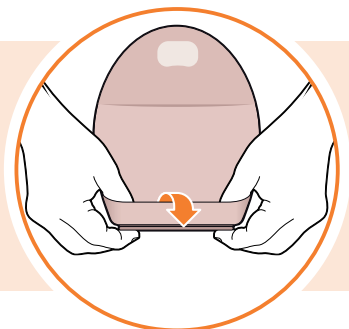


Step 3: Place your thumbs into the Lock-it-Pocket™ and hold the folded end of the tail between your thumbs and your index fingers.

Step 4: Turn the Lock-it-Pocket™ inside out to hide the tail inside the pocket.

2

EMPTYING AND CLEANING THE POUCH

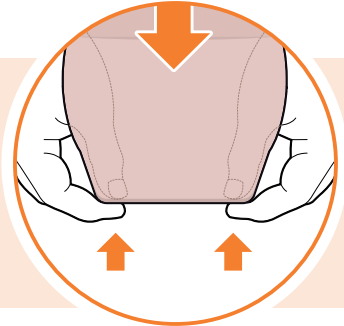


Step 1: Reverse the Lock-it-Pocket™ to reveal the tail.




Step 2: To empty the pouch, tilt the tail up towards your body. Peel open the foam closure and unravel the tail.

Tips for cleaning the tail of the pouch:

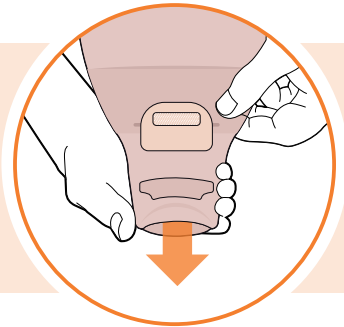

Step 3: Holding the pouch tail with one hand, open the tail by pushing in both ends of the outlet end strips with the other hand to drain the pouch.

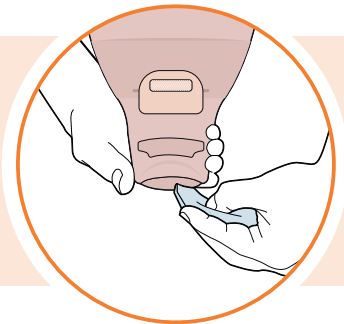

Step 4: Clean the inside surface of the pouch tail with toilet tissue or a wipe. The pouch tail can also be cleaned gently with pH neutral soap and water.

For more information, call us on - 0800 282 254

Please refer to the package insert for complete instructions for use.

TM ® indicates trademarks of ConvaTec Inc. InvisiClose is a registered trademark in the U.S. © 2019 ConvaTec Inc. AP-XXXXXX-XX



Ostomy Care